

Neptune® 2 Ultra

Waste Management System

Neptune 2 Ultra is a new, totally closed **fluid waste management** and **smoke evacuation** system consisting of a Rover and Docking Station. The mobile Rover unit collects surgical waste in the operation room and then quickly disposes of the fluids through the Docking Station with virtually no exposure to healthcare workers. Capable of smoke and fluid evacuation at the same time, Neptune 2 Ultra provides efficient waste collection and disposal while reducing O.R. personnel and patient exposure to surgical fluid.

Powerful Protection for Today's Surgical Environment



IV pole with capacity for (4) 3000cc bags

Large, rotating digital display

4 Liter canister

Single patient disposable manifold

20 Liter canister

2 independently adjustable lines of suction

Smoke evacuator w/ULPA filtration

New coupler connection and cleaning technology

- Large rotating digital display allows for better visibility in the O.R.
- Two independently adjustable suction lines with 2-21 in/Hg range
- Dual canister design – 4L and 20L capacity
- Fluid volume measurement for every 20ml of fluid collected
- Totally closed system protects staff from splash exposure
- New manifold design
- Quiet smoke evacuator with ULPA filter
- Enclosed canister for easy exterior cleaning

NEPTUNE® 2

Neptune[®] 2 Waste Management System



Ultra

Product Specifications

The Neptune Waste Management System consists of a Rover and Docking Station.

Model: 0702-001-000

Neptune Rover Ultra:
Height 70 inches
Depth 23 inches
Width 19 inches

Model: 0702-014-000

Docking Station:
Height 16 inches
Depth 23 inches
Width 23 inches

Installation Guidelines

Plumbing Requirements:

Docking Station should be plumbed to a water connection with a range of 50 to 120 PSI.

Docking Station can be connected to either a wall or floor drain.

Electrical Requirements:

Rover: 120V-, 60 Hz, 12.0A, single phase
Docking Station: 120V-, 60Hz, 3.0A

Service Information

Keep your Neptune 2 Ultra in peak operating condition with our detailed Preventative Maintenance Checklist, exclusive Neptune Docking Detergent and comprehensive ProCare[®] service program.



Preventative Maintenance Checklist

Preventative maintenance is the key to avoiding excessive repair charges, equipment downtime, surgeon irritation, and staff frustration. A comprehensive PM checklist is completed as part of a complete functional evaluation.

Neptune Docking Detergent

Specifically formulated to clean the Stryker Neptune Waste Management System and proven not to cause damage to internal components and seals.

ProCare[®] Service Program

The information presented in this brochure is intended to demonstrate the breadth of Stryker product offerings. Always refer to the package insert, product label and/or user instructions before using any Stryker product. Products may not be available in all markets. Product availability is subject to the regulatory or medical practices that govern individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Products referenced with [™] designation are trademarks of Stryker. Products referenced with [®] designation are registered trademarks of Stryker.

Literature number: 9100-113-000 Rev. None PF/PS

© 2007 Stryker
Printed in the U.S.A.

4100 East Milham Avenue
Kalamazoo, MI 49001 USA
t: 269-323-7700 f: 800-999-3811
toll free: 800-253-3210

www.stryker.com

	Preventative Maintenance (PM) Contract	Maintenance Training Program	Maintenance Training & PM Contract	ProCare*
PM's Proactively Scheduled by Stryker Covers all PM Parts, Labor & Travel Maintenance Checklist provided for your records Reduced Pricing vs. individual PM's	✓ ✓ ✓		✓ ✓ ✓	✓ ✓ ✓
Staff trained for on-site equipment support Service Manual Provided as part of training Save on labor and travel Eligible to purchase repair parts from Stryker		✓ ✓ ✓	✓ ✓ ✓	✓ ✓ ✓
Fixed service costs Covers all Repair parts, labor and travel Eliminate Repair P.O.'s Improved response time due to elimination of P.O. Flexible Payment Options				✓ ✓ ✓ ✓

*With Procare, (1) person is eligible to attend the Neptune Service Training Course held at Stryker.