

Home Page

Stadiometers

- Heightronic 235
- Charder HM200P
- Charder HM200PW
- Charder HM210D
- Ayrton 226
- seca 240
- seca 216
- Measure All 420
- seca 222
- seca 202
- seca 206

Infantometers

Wall Charts

Growth

Charts

235 Heightronic Digital Stadiometer



235 Heightronic Digital Stadiometer

Sale Price: **\$1292.02**
In Stock

[Add To Cart](#)

[Shipping Info](#)

The 235A Heightronic Digital Stadiometer utilizing the same electronic capacitor sensor system scientists use for critical measurements gives a quick digital readout accurate to .01 inch, or .01 centimeters. The 235A Heightronic was selected by the CDC and NHANES IV and is approved for pharmaceutical companies for their studies.

The Heightronic 235A Stadiometer offers superior accuracy and robust design with design features for greater measuring range, ease of use and maintenance-free dependability. The reader / headpiece glides effortlessly along the scale thanks to a more robust bearing system. Just a twist of the handle locks the headpiece where desired and inductive capacitor electronics assure the most accurate digital results. Electronic contacts are located in the back scale to virtually eliminate cleaning and maintenance of the contacts.



Includes: A standard calibration / installation tool to make accurate set-up a snap, Mounting Hardware, Installation and Use Instructions.

Features	
Measurement Range	28-78 inches (70-197 cm)
Graduation	0.01 in (0.01 cm)
Display	Digital LCD
Battery Life	6 - 8 months
Power	Lithium Battery CR123 (included)
Dimensions	58 x 8 x 11 inches (147.32 x 20.32 x 27.94 cm) H x W x D
Return	30 Day Return Policy
Warranty	3 Year Warranty

[Also See](#)

[Heightronic Installation Instructions](#)

[Stadiometers and Height Measurement Devices](#)

[Wall Charts and Growth Tracking](#)

[Growth Tracking and Specialty Measuring Devices](#)

235 Heightronic Digital Stadiometer

[Stud Finding Breakthrough](#)

www.franklinsensors.com

Highest Accuracy Stud Finder Find Studs Faster and Easier



AdChoices

Installation Instructions

It's easy and fast. This can be done by one person in about ten to fifteen minutes. All you'll need in addition to the parts and tool provided is a small level, phillips screw driver and pencil (optional, drill and 1/16 drill bit). Anchors are self tapping no drill needed.

Select a convenient location Use non-carpeted area away from foot traffic (carpet depresses under weight causing inaccurate measurement). Installation on a door or behind a door **is not** recommended.

Mounting: Place the combination **calibration/mounting device** on floor against wall. with pencil in hand rest the scale on the mounting ledge. Use a level to make sure scale is vertical. Mark screw positions (there are four mounting holes, two on top and two on bottom). We recommend predrilling screw holes if you are anchoring to a wall stud. if you do not find a wall stud, simply use the **auger wall anchors** provided. With a phillips screwdriver, screw anchors into place. With holes marked and anchors in place, use **calibration/mounting device** to balance scale on and screw to wall.

Calibration: Now use **calibration/mounting device** as calibration tool. Simply place the tool on the floor about 4 inches from wall (look for decal that indicates **calibrate here**). Slide the head board down the column until it rests lightly on the plastic tool, then press the + or - key until the display matches the number on the calibration tool (note this scale works in inches or centimeters via mode button). Lock display by pressing and holding the ON/OFF key and then quickly press the MODE key, release both at the same time.

You're ready to measure.



[Digital Stadiometer](#)

[Digital Infantometer](#)

[Digital Display Unit](#)

[Operation Instructions](#)

calibration/
mounting device

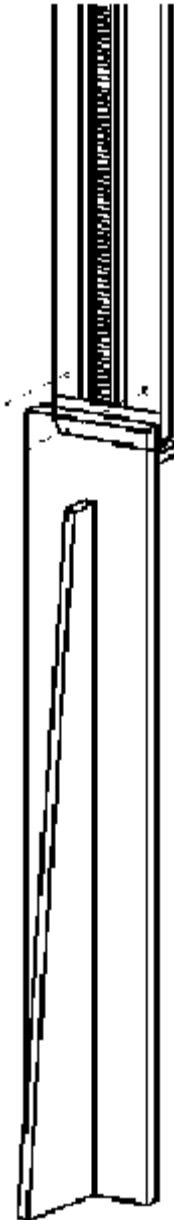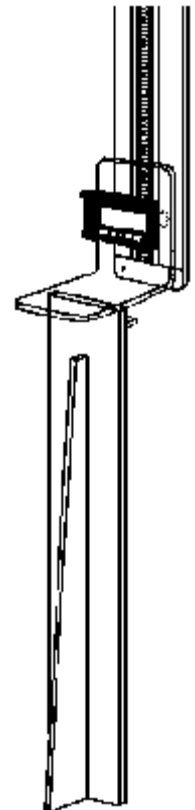

[Ordering Information | Customer Service](#)

stadiometer.com Toll Free 866-500-9808